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**ENGLISH**

**1- INSTRUCTIONS** Brx 3000® is a gel recommended for the atraumatic removal of decayed dentine in teeth and it must be used by professionals.

**2- COMPONENTS** Brx 3000® is a gel composed by Papain 3000 U/mg 10%. Papain is an endo protein similar to the human papain (found in the gastric juices) which has bactericidal, bacteriostatic and anti-inflammatory activities.

**3- INFORMATION FOR DENTISTS** 3.1 STERILIZATION- Product is not sterile. It does not need sterilization. 3.2 CHEMICAL COMPOSITION Every 100ml of the topical gel contains: Papain 3000 U/mg. 10g. Excipients (propylene glycol, citric acid, triethanolamine, sorbitol monophosphate, disodium phosphate, indigotine blue, distilled water, c.s.a 100ml). 3.3 CONTAINERS Brx 3000® comes in 0.5, 1, 0, 3, 0 or 5.0 ml multi-dose syringes or 2, 3, 4, 5 or 6 ml multi-dose bottles. Non-reusable. Their secondary packaging is made of cardboard boxes or polyethylene bags, each containing a unit with its corresponding packaging label. They are packed in cardboard boxes containing 25, 50 or 100 units in them. 3.4 INDICATIONS Brx 3000® is used for the atraumatic removal of caries lesions. 3.5 MECHANISM OF ACTION It acts exclusively over the damaged tissue, since it does not contain an antimicrobial or anti-hipothesis, which prevents the proteolytic action in healthy tissue. Therefore a softening of the damaged tissue is produced that lets the debitter of the damaged collagen fibres, preserving the healthy tissue as it is not demineralized or having no collagen fibres exposed. 3.6 CONTRAINDICATIONS Do not use in any teeth which have any kind of pain, fistula or hyper-sensitivity to any of the components. 3.7 PRECAUTIONS Do not expose the product to direct sunlight or keep in its container. Do not use if the container is damaged or the safety seal is broken. As it is a multi-dose container, it is safe prior after each use. Do not use after the expiration date. Keep out of children's reach. Use blue-edged needles. Handle, assistants and patients should read user's instructions. 3.8 WARNING This product can be handled only by dentists. The empty container is non-reusable. 3. POSSIBLE SIDE EFFECTS There are not side effects known.

**4- SPECIAL CARES** (Information for the patient) It is advisable not to eat or drink for 30 minutes after treatment. This is recommended because of the sealant material not the product itself. Treatment with Brx3000® substitutes the use of rotary tools (turbinas), therefore the patient will not feel pain or distress and they will not need anesthesia or painkillers either. The damaged tissue is removed manually by Brx3000®, so as a alternative which offers greater comfort for patients of all ages.

**5- PACKAGING / CONTAINERS** Brx 3000® is packed in multi dose syringes or bottles. Each ml, can be applied 30 times, treating 15 cases depending on the size of the cavity. The syringes or bottles are identified and labelled with all the information required by ANMAT. This secondary packaging is in a sealed cardboard boxes or polyethylene paper pouches with their label printed on either which contain a unit. Then they are packed in cardboard boxes containing 25, 50 or 100 units.

**6- TECHNIQUE FOR THE DENTISTS**  
*Preparation:* if necessary, widen the cavity with manual tools or - if available - rotary ones until getting an adequate cavity. Apply directly in adequate lesions. Isolate with cotton rolls. Apply Brx3000® over the cavity and wait for 2 minutes. Remove with a blunt-edged spoon remove with pendulum movements, without pressure. Repeat the procedure until.

**HOW TO USE PREVIOUSLY** If necessary, widen the cavity with manual tools or - if available - rotary ones until getting an adequate cavity. Apply directly in adequate lesions. Isolate with cotton rolls. Apply Brx3000® over the cavity and wait for 2 minutes. Remove with a blunt-edged spoon remove with pendulum movements, without pressure. Repeat the procedure until the cavity is free of decayed dentine. 4- necessarily, perform pulp capping, apply a sealant material immediately. Use the written instructions and do not recommend to the patient or to the treatment with Brx3000.

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